ESP

Descripción.

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ENG

Descripción.

Stripline are oscillating strips diamond-coated on both sides for interdental reduction in orthodontics. This is the process known as stripping, which involves creating small spaces between two teeth to allow them to move as necessary during treatment with fixed braces or invisible aligners. In this way, a controlled reduction of the enamel with a final finish is quickly achieved without removal of healthy tooth substance.

Modelos.

6 types of Double Side Strips for IPR.

ML95140-2	Stripline 15sw (15µm/ 0.09mm)	Opener
ML95106-2	Stripline 15 (15µm/ 0.09mm)	Polishing
ML95108-2	Stripline 25 (25µm/ 0.09mm)	Finishing
ML95105-2	Stripline 40 (40µm/ 0.09mm)	Contouring
ML95107-2	Stripline 60 (60µm/ 0.09mm)	Reduction
ML95143-2	Stripline 80 (80µm/ 0.09mm)	Separating

Table 1

Content:

Box of 2 units.

Material.

Stainless Steel (SUS 304) Diamond/Nickel.

Use.

It is connecting to dental hand piece a vertical reciprocating intended to be used in interproximal reduction of teeth. The Striplines are designed to be reused until 5 times.

Indications.

- Initial opening of tight interdental contact points at the beginning of the IPR orthodontics protocol.
- Interproximal enamel reduction during the IPR orthodontics protocol.
- Finishing and polishing of the interproximal Surface.

Contraindications.

- Crowding more than 8 mm per arch.
- Poor oral hygiene.
- Active periodontal diseases.
- Enamel hypoplasia.
- Hypersensitivity to cold.
- High caries index, and multiple restorations.
- Rectangular-shaped front teeth.
- Round-shaped premolars.
- Young patients with large pulp chambers.
- Avoid applying loads greater than 100 grams as this could cause damage to the instrument and/or the tooth.

How to Use.

1. Open up the product package and check whether the opened product has any abnormality, impurity or foreign substance and if any, it should not be used.
2. How to use the product should be thoroughly understood through Instructions for Use.
3. Various sterilized devices necessary for surgical procedure should be prepared.

- As this product is nonsterile, it should be sterilized before use.
 - Sterilizing method : High pressure steam sterilization (ISO 17665-1,2)
 - Temperature: 134°C
 - Duration: 15 minutes
 - Pressure: 2.1 bar
 - Drying duration : 30 minutes.
- Prepare a low-speed dental handpiece with the Stripline.

Recommendations for Use.

- Apply StripLine 15sw opener at beginning of the IPR orthodontics protocol for the opening of tight interproximal contact point.
- Subsequently apply StripLine according to the IPR protocol for orthodontic treatment.
- In case of very tight contact points separate the teeth by placing a wooden wedge cervically. This make stripping more comfortable both for operator and patient. Set the micro motor at 10,000 RPM/min max. with OK handpiece this correspond to 2500 RPM/min.
- Active water spray during the entire treatment (minimum 50 mL/min).
- Regulate water flow in case of water excess or shortage.
- Insert the strip in a cervical direction while the contra angle is moving. This prevents the instruments from jamming.
- Do not stop the strip in the interdental space.
- Remove the strip from the interdental space whilst the contra angle is moving.

Storage Conditions.

- Keep in packaged status in an indoor cabinet (at room temperature).
- Keep at room temperature (1~30°C) and avoid direct sunlight.

Side Effects.

- Caries and periodontal disease in teeth.
- Excessive heat is known to cause damage to the pulp of the tooth.

Warnings.

- This product should not be used for any purposes other than its specified intended use.
- As this product is nonsterile, it should be sterilized before use.
- This product can be reusable; when re-use the product, it should be sterilized before use.

Precautions

- Do not use any product that has abnormality or foreign substances.
- The Stripline may become loose due to inappropriate insertion upon its placement.
- This product is a product sterilized by user; it should be sterilized before use in accordance with specified sterilizing method and should be immediately used after sterilization.
- Excessive lateral swerve or obstruction during the oscillation movement increases the risk of instrument damage.
- Stop using the product immediately and contact us if there is any problem with product appearance.
- Please check model and size before use the strip.
- Disposal of products must comply with the relevant regulations for medical waste in your country.

Internacional Ventur

Internacional Ventur, S.A.

P. I. Ciudad del Transporte · C/ Luxemburgo, 75
+34 964 252 552 · 12006 Castellón · Spain
www.iventur.com

JAINEK CO., LTD

#541 Galmaesunhwan-ro 166beon-gil 46,
Guri-si, Gyeonggi-do, Republic of Korea

KTR EUROPE GMBH

Mergenthalerallee 77 Eschborn, 65760,
Germany.

www.medicaline.es

ML951XX-2 I2405161600